

allard^{INT}

SELECTION[®]

ULNAR DEVIATION SPLINT

Improve finger alignment to enhance hand function and functional grip.



- Low profile design
- Meets the shape and contours of the hand
- Breathable fabric
- Fabric can be trimmed
- Easy fastening
- Can be worn with Selection Wrist Splint
- Machine washable

Distributed in Canada by:



Professional Orthopedic Products

Toll-Free: 1-866-334-4441
csr@poportho.com
www.poportho.com

SELECTION[®] ULNAR DEVIATION



SELECTION[®] Ulnar Deviation Splint is intended to improve finger alignment and enhance hand function and functional grip.

This hand splint is exclusively designed and produced using our unique three layer Selection fabric. With its low profile design, it meets the contours of the hand, without impeding function. The fabric can be trimmed without fraying, and therefore allows further customization if required. The easy fastening surface allows the hook to be placed anywhere on the splint to optimise the directional pull needed. The orthosis is designed so that the material extends far to the ulnar side at 5. MCP to support the best possible direction of pull and distribute the pressure.

The Ulnar Deviation splint supports the arches of the hand whilst providing re-alignment, stabilisation and can contribute to the management of pain.

Should you need to provide wrist stabilisation for your patient, this splint can also be worn in combination with a Selection wrist splint. Specially designed reinforcements of Low temperature plastic (non-coated) adhere directly to the orthosis if needed.



Intended use

For patients with inflammatory and degenerative diseases, such as rheumatoid arthritis, that has resulted in ulnar deviation. Selection Ulnar Deviation can also be used for hypermobile joints and weak muscles to provide stability. Not intended for use with fixed contractures in the hand.



Item No. Left	Item No. Right	Size	Circumference MCP 2-5
35218 1010	35218 2010	X-Small	14 - 17 cm
35218 1011	35218 2011	Small	17 - 20 cm
35218 1012	35218 2012	Medium	20 - 23 cm
35218 1013	35218 2013	Large	23 - 27 cm

Campscandinavia, JULI_2021 ©

Distributed in Canada by:



Professional Orthopedic Products

Toll-Free: 1-866-334-4441
 csr@poportho.com
 www.poportho.com

Manufactured by:

ALLARD INTERNATIONAL
 Camp Scandinavia AB
 Karbingatan 38
 SE-254 67 Helsingborg SWEDEN
 info@allardint.com